

# Supplies

1. New dressing

2. Skin care supplies



Perform hand hygiene.



Remove new dressing from packaging and set aside.



Secure neck flange while slowly removing the old dressing, one side at a time.



Inspect skin and stoma for signs of infection, irritation, and skin breakdown.



Perform skin and stoma care. Wipe skin around stoma and under the neck flange using e.g. Tracoe Cleaning Cloth. Allow skin to dry.



Place new dressing between the neck flange and skin with the slit on the dressing facing toward your chin.

Discard the old dressing.



Watch video instructions here



# TRACOE

## TIP #1

The dressing should be changed at least every 24 hours, or if needed such as when soiled due to secretions or after a cleaning procedure.

#### TIP #2

Inspect the condition of the skin under the dressing during changes for signs of infection, irritation, and skin breakdown. Report any findings to your healthcare provider who can discuss which dressing options are best suited for your skin care needs.

### TIP #3

In certain situations such as excess secretions or skin irritation, it may be helpful to apply a film forming barrier such as Provox Skin Barrier which provides a protective layer between the skin and dressing. Apply Provox Skin Barrier after cleaning the skin and before applying new dressing.

**Head office:** Atos Medical AB Hyllie Boulevard 17

Hyllie Boulevard 17 SE-215 32 Malmo, Sweden Tel: +46 (0) 415 198 00 Email: info@atosmedical.com **Manufacturer:** Atos Medical AB Kraftgatan 8 SE-242 35 Hörby, Sweden Manufacturer: Tracoe Medical GmbH Reichelsheimer Straße 1/3 55268 Nieder-Olm, Germany

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.

The information contained in this material does not replace the Instructions for Use of the products demonstrated. You must see your physician, clinician, or qualified caregiver for advice on your condition and on products that may be appropriate for you.

MC3017-TcEN 202305

