

Supplies

1. Clean inner cannula
2. Gauze dressing

Changing and cleaning your inner cannula is an essential part of tracheostomy care. This helps prevent blockage, removes mucus plugs, and reduces infection that can occur. Your inner cannula should be removed, cleaned, and changed at least daily and more often as needed.



Perform hand hygiene.



Prepare a cleaning station. Place a clean inner cannula in front of you to the left. This will be your "clean" side.

Place a clean gauze dressing in front of you to the right. This will be your "dirty" side.



Secure neck flange while removing the inner cannula from the outer tube.



Place the used inner cannula on the gauze dressing on the "dirty" side.



Secure neck flange while inserting the clean inner cannula into the outer tube. Ensure the inner cannula is locked in place.



Watch video
instructions here

Head office:
Atos Medical AB
Hyllie Boulevard 17
SE-215 32 Malmö, Sweden
Tel: +46 (0) 415 198 00
Email: info@atosmedical.com

Manufacturer:
Atos Medical AB
Kraftgatan 8
SE-242 35 Hörby, Sweden

Manufacturer:
Tracoe Medical GmbH
Reichelsheimer Straße 1/3
55268 Nieder-Olm, Germany

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.

The information contained in this material does not replace the Instructions for Use of the products demonstrated. You must see your physician, clinician, or qualified caregiver for advice on your condition and on products that may be appropriate for you.

MC3019-TcEN 202305