

Supplies

- 1. Tracoe Cleaning Powder
- 2. Tracoe Cleaning Tub
- 3. Swabs
- 4. Gauze dressing

Inner cannulas should be cleaned at least daily and more often as needed to remove and prevent mucus build up and keep tube surfaces clean.



Prepare a cleaning area.



Mix one scoop (about 5 g) Tracoe Cleaning Powder with 500 ml tap water and place in the Tracoe Cleaning Tub.



Place the inner cannula in the mesh basket and immerse in cleaning solution.



Remove the mesh basket after soaking for 10-15 minutes.



Remove any remaining mucus on the inner cannula with a swab as per instructions.

Discard swab after use.



Rinse inner cannula in a clean bowl of water or under lukewarm tap water.

Remaining solution can be reused up to 24 hours.



Leave to air dry.

Place in clean and dry storage container until next use.



Watch video instructions here





Head office:

Atos Medical AB Hyllie Boulevard 17 SE-215 32 Malmo, Sweden Tel: +46 (0) 415 198 00 Email: info@atosmedical.com

Manufacturer: Atos Medical AB Kraftgatan 8 SE-242 35 Hörby, Sweden Manufacturer: Tracoe Medical GmbH Reichelsheimer Straße 1/3 55268 Nieder-Olm, Germany

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.

The information contained in this material does not replace the Instructions for Use of the products demon-strated. You must see your physician, clinician, or qualified caregiver for advice on your condition and on products that may be appropriate for you.

MC3022-TcEN 202305

Breathing-Speaking-Living atosmedical.com