
# PROVOX<sup>®</sup> Adhesives

## Instructions for Use

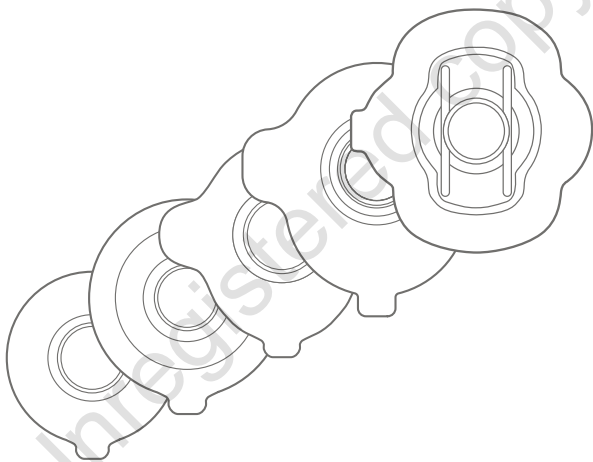
# Atos


Atos Medical **Your voice**

# 1. StabiliBase, StabiliBase OptiDerm


1.1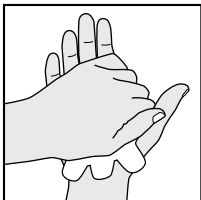


1.2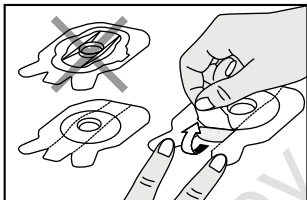


1.3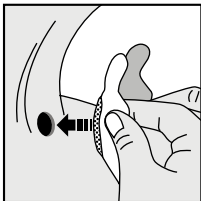


1.4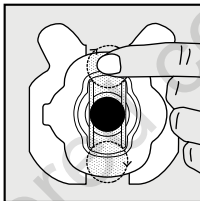


1.5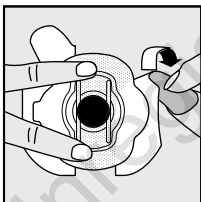


1.6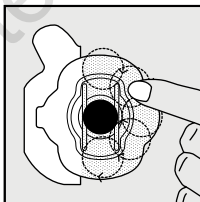

1.7

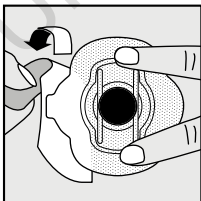
1.8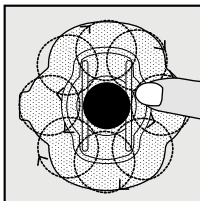


## 2. FlexiDerm, Regular, XtraBase, OptiDerm


2.1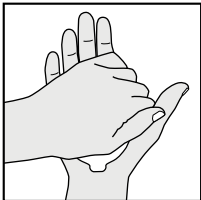


2.2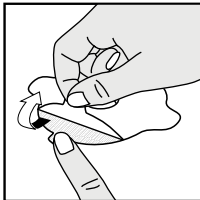

2.3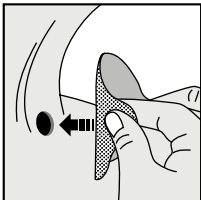

2.4

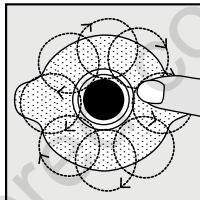## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Patents and trademarks**

Provox® and Provox® XtraBase® are registered trademarks owned by Atos Medical AB, Sweden. FlexiDerm™ and OptiDerm™ are trademarks of Atos Medical AB.

Skin Tac™ is a trademark of Torbot Group.

For information about patents, please refer to the webpage [www.atosmedical.com](http://www.atosmedical.com).

## Contents

|                            |    |
|----------------------------|----|
| ENGLISH .....              | 6  |
| DEUTSCH .....              | 9  |
| NEDERLANDS.....            | 12 |
| FRANÇAIS .....             | 15 |
| ITALIANO.....              | 18 |
| ESPAÑOL .....              | 21 |
| PORTUGUÊS.....             | 24 |
| SVENSKA .....              | 27 |
| DANSK.....                 | 30 |
| NORSK.....                 | 33 |
| SUOMI .....                | 36 |
| MAGYAR .....               | 39 |
| POLSKI.....                | 42 |
| ΕΛΛΗΝΙΚΑ .....             | 45 |
| TÜRKÇE .....               | 48 |
| РУССКИЙ .....              | 51 |
| 日本語 .....                  | 54 |
| 한국어 .....                  | 57 |
| עברית.....                 | 62 |
| العربية.....               | 65 |
| ORDERING INFORMATION ..... | 66 |

The Instructions for Use, which accompanies this product, may be revised from time to time and must therefore be reviewed prior to each procedure in which the product is used.

## Intended use

The Provox Adhesives are single use devices intended for laryngectomized patients breathing through a tracheostoma. The devices are attached to the skin around the tracheostoma in order to provide attachment of components of the Provox HME System.

## Description of the device

Provox Adhesives are designed to ensure an airtight attachment for the Provox HME system components.

The adhesives have an adhesive part, a peel-off liner and an adapter where components of the HME system can be connected. The adhesive part is made of hypo-allergenic materials of three different types described below.

### FlexiDerm adhesives

FlexiDerm is a very flexible material and has the strongest adhesive properties. It is a sticky, yet soft and flexible adhesive.

#### The FlexiDerm adhesives are:

- **Provox StabiliBase:** Has a specially designed firm base with vertical stabilizing bars that provides support to the tracheostoma during speech. It is especially suitable for deeper tracheostomas and an ideal choice when using Provox FreeHands HME. The peel-off liner is divided into three sections for easier application to the skin.
- **Provox FlexiDerm Round/Oval/Plus:** This adhesive is especially suitable for patients who prefer a sticky, yet soft and flexible adhesive.
- **Provox XtraBase:** Has a convex shaped base. Designed to facilitate and improve skin attachment for deep tracheostomas and/or if a Provox FreeHands HME is used.

### OptiDerm adhesives

The hydrocolloid material of the OptiDerm adhesives is a substance that forms a gel with water. The gel itself functions as glue when pre-warmed between your hands before applied to the skin.

### **The OptiDerm adhesives are:**

- **Provox StabiliBase OptiDerm:** Has a specially designed firm base with vertical stabilizing bars that provides support to the tracheostoma during speech. It is especially suitable for deeper tracheostomas and an ideal choice when using Provox FreeHands HME. The peel-off liner is divided into three sections for easier application to the skin. The OptiDerm material is suitable for patients with sensitive skin.
- **Provox OptiDerm Round/Oval/Plus:** Intended for patients with a sensitive skin or with skin that is not yet healed properly, e.g. after surgery and/or irradiation. It can be applied gently on the 1st postoperative day.

### **Regular adhesives**

The Regular material is a film, less flexible and with slightly less adhesion strength than the FlexiDerm material.

### **The Regular adhesive:**

- **Provox Regular Round/Oval/Plus:** Mainly intended for persons with a regular tracheostoma and skin.

## **WARNINGS**

- The adhesive may irritate the skin. Stop using the adhesive if skin irritation develops and consult your clinician.
- Do not use the Provox Adhesives during radiotherapy that is delivered to the area covered by the adhesive. Consult your clinician before you resume use of the adhesive after radiation therapy.
- Reuse of the adhesive, by yourself or someone else, may cause transfer of micro-organisms which can lead to infections.
- Reuse may also cause reduced efficiency of the adhesive properties which can lead to air leakage during speech and reduced efficiency of the attached HME.
- Only use genuine Provox system components that are intended for use with Provox Adhesives. Other devices may cause personal injury or damage to the products.

## **Instructions for use**

### **Preparation**

Inspect that the adhesive is not damaged or has tears in the adhesive or around the base plate.

Make sure the skin is clean and dry. If needed, you can apply skin protection products, like Provox Skin Barrier or Skin Tac™ or adhesive supporting products such as Provox Silicone Glue (read the Instructions for Use accompanying the products).

If you use OptiDerm/StabiliBase OptiDerm adhesives, pre-warming the adhesive between your hands improves the adhesion.

## **Application**

### **For instructions on how to apply:**

- Provox StabiliBase and Provox StabiliBase OptiDerm please see pictures 1.1 – 1.8.
- Provox FlexiDerm, Regular, XtraBase and OptiDerm please see pictures 2.1 – 2.4.

## **Removal**

Replace the adhesive when it has become loose or dirty. Peel the adhesive off gently, using the finger lift tab.

An adhesive remover, e.g. Provox Adhesive Remover, may be helpful for removing adhesives or glue (Read the Instructions for Use accompanying the product). Always clean the skin with a Provox Cleaning Towel and/or soap and water after use of an adhesive remover. Dry the area carefully.

## **CAUTION:**

- When cleaning the skin from residual glue, prevent particles/fluids from entering the tracheostoma.
- When using OptiDerm in the postoperative period or on sensitive skin, it should be removed very slowly and carefully.

## **User assistance information**

For additional help or information please see the back cover of this Instructions for Use for contact information.

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© Atos Medical AB, 2017.