



PROVOX[®] Flush

ATOS

MEDICAL

Leading the way in ENT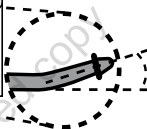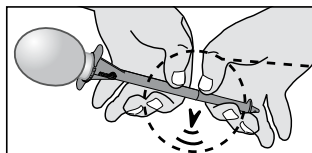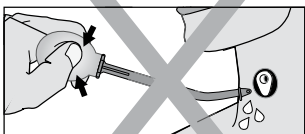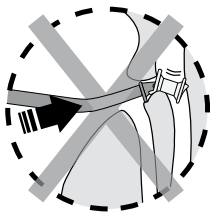

Max 25°

Diagram illustrating the correct use and disposal of the device.


The device is shown in its packaging. An arrow points to a calendar grid showing the first 30 days of the month.

1	2	3	4	5	6	7
8	9	10	11	12	13	14
15	16	17	18	19	20	21
22	23	24	25	26	27	28
29	30					



x 12


Max use 12 months


The device is shown being disposed of in a trash can.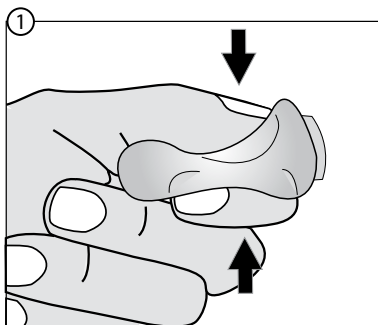

+

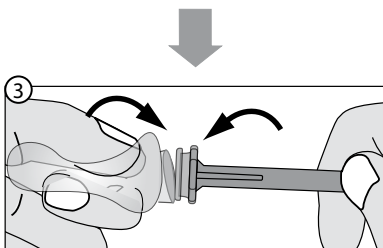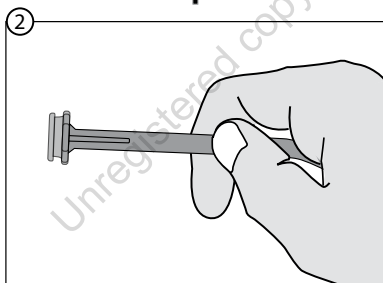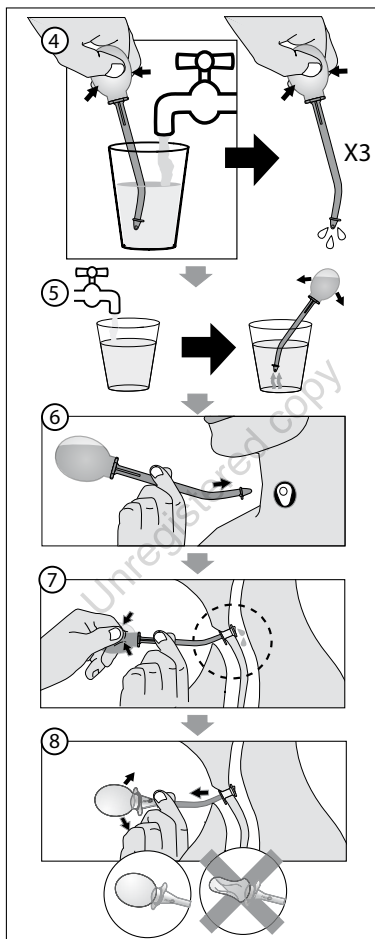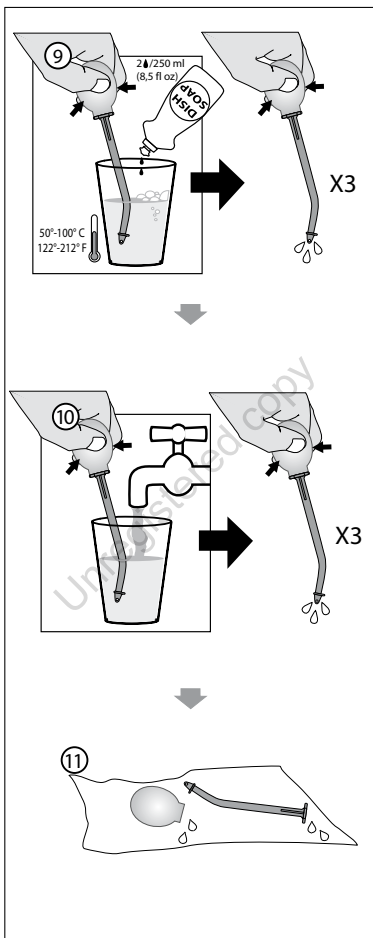
⑫

x1

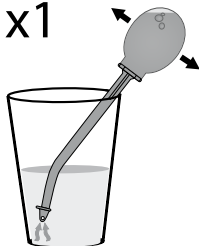70 % Ethanol

10 min

or

70 % Isopropylalcohol


10 min

or

3 % Hydrogenperoxide

60 min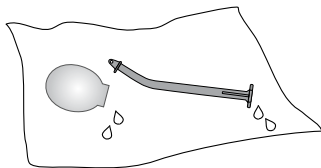

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

The Provox® System is protected by several patents. Provox® is a registered trademark owned by Atos Medical AB, Sweden.

Unregistered copy

Contents

ENGLISH	9
DEUTSCH	11
NEDERLANDS	13
FRANÇAIS	15
ITALIANO	17
ESPAÑOL	19
PORTUGUÊS.....	21
SVENSKA	23
POLSKI.....	25
РУССКИЙ	27
日本語	29
한국어	31
עברית	34
العربية	36

The manual, which accompanies this product, may be revised from time to time and must therefore be reviewed prior to each procedure in which the product is used.

Intended use

The Provox Flush is intended to be used to flush drinking water or air through the inner lumen of a Provox voice prosthesis for cleaning purposes. The Provox Flush is intended for both home and clinical use by patient or clinician.

WARNINGS

The flush is only recommended for patients with sufficient manual dexterity acceptable vision and satisfactory cognitive ability.

Cleaning and disinfection

Clean the Provox Flush after each use according to fig. 9-11.

Do not use any water other than drinking water to clean and rinse the device.

Disinfect the Provox Flush at least once a day (fig. 12) with one of the following methods:

- Ethanol 70% for 10 minutes
- Isopropylalcohol 70% for 10 minutes
- Hydrogenperoxide 3% for 60 minutes

Caution: Do not clean or disinfect by any other method as this might cause product damage and injury.

If the Provox Flush looks dirty or has air dried in an area with a risk of contamination the device should be both

cleaned and disinfected before use. A risk of contamination could exist if the device has been dropped on the floor or if it has been in contact with a pet, someone with a respiratory infection, or any other gross contamination.

During hospitalization it is important to both clean and disinfect the Provox Flush after use but also **before** use since there is an increased risk of infection and contamination. In a hospital it is better to use sterile water for cleaning and rinsing, rather than drinking water.

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© Atos Medical AB, Sweden. Edition 201306A, REF 10055