# TrachPhone®

## Instructions for Use

### Trach-HME with Built-in Speech Valve

TrachPhone® is a registered trademark owned by Atos Medical AB, Sweden.  
TrachPhone® is protected by patent SE 512 029 and US 6 422 235.

**ATOS**  
MEDICAL

Leading the way in ENT

Atos Medical AB,  
Kraftgatan 8, P.O. Box 183, SE-242 22 Hörby, Sweden  
Tel: +46 (0)415 198 00 • Fax +46 (0)415 198 98  
www.atosmedical.com • info@atosmedical.com

© Atos Medical AB, Sweden. Edition 201310A, REF 90749

(EN)

### Indications

For patients breathing spontaneously via an ET tube or a tracheostomy tube in the hospital or at home.

### Technical data

|                                |  |
|--------------------------------|--|
| Recommended tidal volume range | 50 - 1000 ml   |
| Compressible volume            | 9.5 ml (dead space)  |
| Moisture output*               | VT= 250 ml: 23.4 mg/l<br>VT= 500 ml: 20.5 mg/l<br>VT= 1000 ml: 16.6 mg/l |
| Resistance to flow             | 35 ± 15 Pa at 30 l/min   |
| Weight                         | 2.9 g  |
| Length                         | 22 mm  |
| Height                         | 28 mm  |
| Width                          | 38 mm  |
| Non-conductive                 |  |

\*MDA Report 01052

### Connections

15 mm female connector for ET tube or tracheostomy tube. 4 mm connector for supplemental oxygen supply. Use appropriate size of oxygen tubing. The TrachPhone has a valve for forced inspiration/expiration or suctioning.

### Instructions for use

The device should be placed at the open end of the ET tube or tracheostomy tube. Check that the connection is properly fixed. If supplemental oxygen is required, connect an oxygen tube to the oxygen port. Normal efficiency is reached after a few breaths. Should be replaced every 24 hours or as required, to avoid resistance increase due to accumulation of secretions.

Before use, check product for deformity and valve function by closing the valve and releasing it. Do not use if the device shows any signs of defect. Do not use if the package is opened or damaged.

Store in clean, dry, dark, temperate conditions.

### Speech valve

The TrachPhone contains a valve with a spring that can easily be occluded with a finger facilitating speech. After releasing the finger, the valve will open automatically.

### Contraindications

Do not use beyond recommended tidal volume range, as the added dead space may cause CO<sub>2</sub> retention at too low tidal volumes. A too high tidal volume may lead to unsatisfactory humidification.

Do not use on dehydrated patients or patients with very heavy secretions from the lungs and airways.

### Caution

Single use only: do not sterilize or reuse, as the intended function will be affected, and this might increase the risk of possible transmission of infection. Destroy after use.

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