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*Single-use inner cannulae*

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**UK** Instructions for use

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 Packaging unit

**Purpose:**

Single-use inner cannulae are used in tracheal tubes for a daily change and where cleaning and disinfection are not planned. This can be established as an in-house hygiene requirement in home-care or hospitals. To meet these requirements correctly, inner cannulae are used, which after use are discarded and replaced.

**Description:**

The single use inner cannula is inserted into the outer tube and fixed in position using the click-closure. They feature both a universal connection and a speech valve connection to facilitate fixation of an HME (heat-moisture exchanger) or a speech valve. The surface of the inner cannula is lubricated with a preserving oil (e.g. **PRIMA-STOM II oil** from HEIMOMED) to facilitate insertion.

**Packaging:**

10 inner cannulae per pack

**Safety Information:**

As is otherwise customary, single use inner cannulae are not made to fit the outer tube, as they are supplied separately. It is therefore possible that inner and outer tubes do not fit together with 100 % tightness. Single-use inner cannulae may not be used with length-optimised tracheal tubes. Inserting an inner cannula into a tracheal tube with optimised curvature can be a cumbersome process. Single-use inner cannulae are always supplied non-sterile.

If a serious adverse event occurs involving the single-use inner cannulae, this must be reported to the manufacturer and the competent authority.

**Storage:**

These products must be stored at room temperature in a clean, dry environment.

**Product disposal:**

Disposal of products must be carried out in compliance with current national regulations applying to waste products.

**Caution:** Warranty, guarantee and liability claims against the manufacturer will be nullified if any changes are made to the product by users or if the product is used outside of the scope of applications described in these instructions.