Sprechventil SILKO-VENT SOFT / UNI

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UK Instructions for use

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Explanation of Symbols:
 Packaging unit
Purpose:

The **SILKO-VENT SOFT / UNI** speech-valve provides finger-free speech for tracheotomy patients. Here the tracheostomy does not need to be closed with the finger to override the airway, as the airway is diverted over this valve. **SILKO-VENT SOFT / UNI** speech-valves are for use by one patient only.

Description:

- 1: Connecting speech-valve to cannula
- 2: Connecting speech-valve to HME (Moisture/Heat-Exchanger) and cannula with attachment ring / universal attachment
- 3: Connecting speech-valve to cannula with universal connector
- 4: Connecting speech valve to HME (moisture/heat exchanger) and cannula using the universal connector

Cleaning:

Cleaning may be carried out by placing for 2-3 minutes in clear, lukewarm water. Allow the product to dry in the air.

Product Disposal:

Disposal of the product may be carried out only in accordance with current national regulations for waste materials.

Storage conditions: Cool and dry.

Caution: Warranty, guarantee and liability claims against the manufacture are nullified if changes are made to the product by users, if the product is used outside of the scope of applications described in these instructions for use or if the cleaning instructions are not complied with.

The product may be used for a maximum of 6 months. After that, it must be replaced by a new product.

Safety Information:

SILKO-VENT SOFT / UNI speech-valves are intended exclusively for tracheostomized patients and may not be used for laryngectomized patients. Malfunctions may result from blockage of the valve with secretion. In such cases, remove and clean the valve. If this does not remedy the malfunction, the speech-valve must be replaced. The valve must not come in contact with oil, as this can in time affect its functional efficiency. There is a risk of cross-contamination if the **SILKO-VENT SOFT / UNI** is used by more than one patient.

Contraindications:

Speaking valves are intended exclusively for tracheostomized patients.

Speaking valves shall not be used:

- By laryngectomized patients.
- By patients that are not able to exhale through the upper airways.
- By patients unable to remove or operate the device, unless the patients are under constant supervision of a clinician or a trained caregiver. For example: Patients who are unable to move their arms, patients with decreased level of consciousness, or patients with diseases that put them at risk for unpredictable periodic loss of consciousness, or children that do not have the appropriate mental capacity to handle the device, which may lead to restricted breathing or suffocation.
- By patients not tolerating added breathing resistance.
- In combination with tracheostomy tubes that does not allow sufficient airflow through or around the tube.
- In combination with endotracheal tube.
- In combination with a tracheostomy tube with a self-inflating or foam-filled cuff.